BEAUTECH

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TECH E DIGITAL

SOMMARIO SUMMARY

Area TECH

TECH Area

OneNeed.Evo	06
Be-Face	10
Be-Face H2O	14
Be-Test	18
Be-Check	20
Be-Thermocheck	22
Be-Complex	24
Be-BodyLaser.Pro	28
Be-Lymphopres	32
Be-BodyTronic.Pro	36
Be-Matrix	40
Enbio	44
Be-Motion	48
Be-Infrastep.Pro	52
Be-Cryo.Pro	56
Be-Circle.Pro	60
Be-Gravity.Pro	64
Collagenium	68

01

Area DIGITAL

DIGITAL Area

Be-Solution in cloud	90
Beauty App	94
Beauty Online	96
Beauty Social	98

02

Focus POINT

BEAUTECH E' TECH E DIGITAL

BEAUTECH IS TECH AND DIGITAL

Innovazione e Performance
Innovation and performance

Solarium Collagenium antiage
Solarium Collagenium antiage

Design moderno e funzionale
Modern and functional design

Software gestionale in cloud e sito web dinamico
Management software in cloud and dynamic website

Device multifunzione viso e corpo
Multifunction face and body device

Creazione shop online e Beauty App
Creation of online shop and Beauty App

Rigorosamente Made in Italy
Strictly Made in Italy

Supporto Social media marketing
Social media marketing support

Rispetto delle normative vigenti e agevolazioni fiscali
Compliance with current regulations and tax benefits

Campagne di Lead generation
Lead generation campaigns

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Italian device with two international patents for face, body and semi-permanent make up aesthetic treatments. A real technological concentrate that, thanks to a single handpiece, signs all beauty solutions: wrinkles, dilated pores, skin spots, scars, loose skin, stretch marks, lips or eyes off and thinned hair or eyebrows. Thanks to the power of the engine and the internal pushing mechanism of the cartridges, the movement is precise and vibration-free to meet every need. The device presents an intuitive and sophisticated graphic design with a color touchscreen HD display that can easily go with the choice of different preset programs dedicated to each technique and treatment. It features an incredibly durable and elegant aluminum case.

BE-NEED.EVO | ONENEED.EVO
BE-NEED.EVO | ONENEED.EVO

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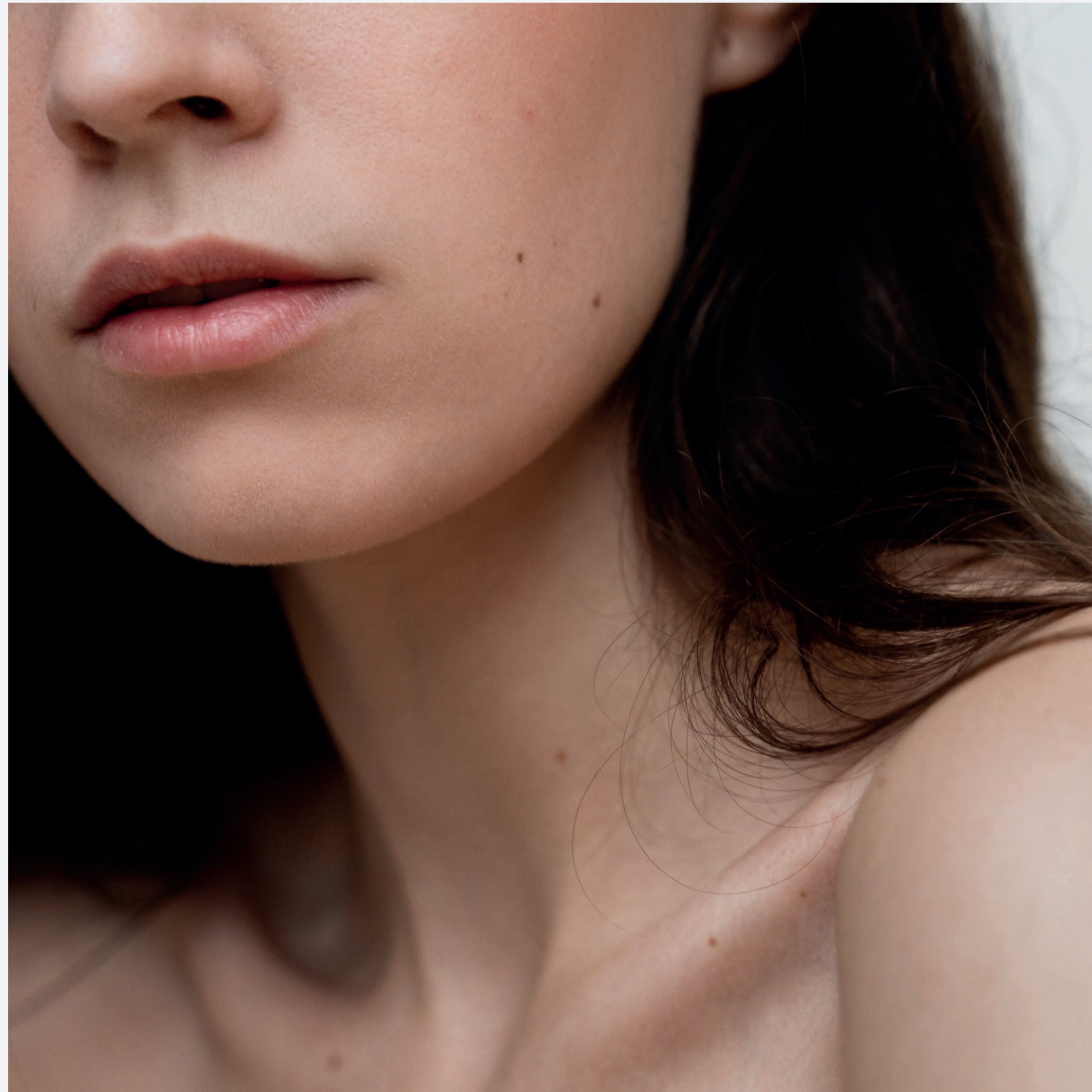
RF NEED SYSTEM: Innovative and patented cartridge that simultaneously combines the stimulation of Aesthetic Pricking³ to the stimulation of Bipolar Radio Frequency. Thanks to this technological concentrate it is possible to obtain a real skin rejuvenation, a cellular regenerative stimulation that gives new life to the skin.

AESTHETIC PRICKING®: Aesthetic treatment for the mechanical bio-remodeling of the skin. The point applicators in the specific cartridge create thousands of non-invasive micro stimulation in the epidermis. The skin reacts to these "micro stimulation" promoting tissue regeneration.

NANOPEEL TREATMENT: Aesthetic treatment that combines the exfoliating action of hydroxyacids to the mechanical scrub carried out by the specific cartridge with 130 rounded nano-tips designed specifically to smooth the skin and make it brighter and more homogeneous.

SEMI-PERMANENT TATTOO: Collection of cartridges for the treatment of semi-permanent makeup (micro, trico and areola pigmentation). A personalized make-up that not only enhances the features, but also corrects some imperfections of face and body.

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Be-Face is the system designed to take care the imperfections on the face, neck and décolleté. A device that contains the most innovative technologies for cleaning, oxygenation, toning and regeneration treatments.

The special handpieces can be used in a synergistic way or can be used also individually. Be-Face and Be-Face.Pro allow to obtain excellent results as far as wrinkles, skinspots, loose skin and impurities.

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HYDRATION CHECK-UP: The monitor shows the image of a face on which are indicated the points where to place the handpiece for the detection of the average value of hydration.

DERMO INFUSION: Dermo Infusion, through the emission of dynamic electrical impulses, helps to open channels in the dermis to convey in depth any type of functional substance applied, chosen according to the imperfection to be treated.

ULTRASOUND CLEANING: Performs deep cleaning treatment sessions, surface cleaning for stains, death cell removal and stimulation of cell replacement.

MICRODERMABRASION: Microdermabrasion is used to reshape the surface of the skin and consists of mechanical abrasion of damaged superficial layers of the epidermis, removal of dead cells and stimulation of cell turnover.

VEHICLE ULTRASOUND: Able to reach the deepest layers of the tissue, it conveys the cosmetics chosen according to the imperfection to be treated, stimulating the reactivation of the microcirculation and gradually reducing wrinkles, resulting in skin toning.

RADIO FREQUENCY: This treatment with an extraordinary lifting effect counteracts skin relaxation. Raising the temperature of the existing connective tissue and activating, through heat, fibroblasts to produce new Collagen and new Elastin.

DERMOCAMERA: Determination of the type of facial skin, comedones, hydration to evaluate which strategies to adopt for each single blemish, in terms of product correct for home use or custom protocol of specific treatments in the Institute.

CHROMO MASK: Chromo wellness treatment that uses different colors customizable according to individual preferences, thus achieving greater relaxation and benefit in the phase of laying the cosmetic mask applied during treatment.

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Be-Face H2O represents the modern evolution of traditional facial cleansers, revolutionizing the way to take care the skin. It uses an innovative approach that combines: hydradermoabrasion to the action of hydroxy acids peeling, the infusion of specific active agents to the oxygenating action of vacuum massage and muscle stimulation of Tonic Wave to the diathermy of radiofrequency. All in one procedure. It is not a simply skin cleaning, but a deep regeneration. Be-Face H2O becomes the essential foundation for healthy and radiant skin.

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SKIN ANALYSIS: Professional digital visual assessment tool that allows you to identify your skin condition, with maximum simplicity and share images with others.

H2O DERMA PEEL: Aesthetic treatment that combines mechanical hydradermabrasion with the exfoliating action of the Hydroxyacid solution to smooth the skin of the face and make it brighter, more homogeneous.

H2O DERMA INFUSION: Aesthetic treatment that combines the oxygenating action of H2O Vacuum massage with the infusion and delivery of specific activities thanks to the synergistic emission of Derma Infusion for the restoration of the skin barrier and to regenerate and revitalize the skin deeply.

ULTRA SOUND WAVE: Deep conveying treatment of the active contained in the applied masks, improving and refining the aesthetic results.

COLLAGEN TONIC WAVE: It simultaneously combines Tonic Wave muscle stimulation with Radiofrequency stimulation. Thanks to this synergy, it is possible to improve the condition of elastic fibers, oxygenate and tone facial expressions, and invigorate the microcirculation to redefine facial contours.

CHROMO MASK: Advanced device that uses Bio-LED technology to offer a pose treatment for relaxation and well-being.

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Be-Test.Go is Beutech's compact dermocamera dedicated to facial skin analysis. Through magnification, measurement and guided visual evaluation, it allows professionals to identify key skin conditions and guide the selection of:

- home skincare routines
- personalised professional treatment protocols

Compact, intuitive and highly efficient, it integrates seamlessly into daily aesthetic practice, enhancing the quality of consultation.

Be-Test.Go strengthens professional credibility and enhances treatment recommendations.

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Be-Check is the new Beatech impedancemeter, a sophisticated and easy-to-use instrument that allows you to measure the impedance of the human body through which it is possible to break down a person's weight into lean mass, fat mass, intra and extracellular water both overall and segmentally. The impometric test is an essential guide to weight management through the data provided by the Report.

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Contact thermography is based on the use of micro-encapsulated liquid crystal plates that have the property of changing color as the temperature changes. By placing these plates on the body it is possible to detect the skin temperature (which reflects what happens in the underlying fabrics) and therefore highlight, through color images, the presence or absence of various stages of cellulite and the type of adiposity on various parts of the body.

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Many results in a single console for face and body

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Beautech, is always near to each Beautician, and provides practical and real answers to each requirement as far as "Beauty" with excellent technologies studied to act in a precision way for each problem. Beautech makes available two different Machines, all system of last generation that are a mix of different technologies, so the operator can work about different requests. The special handpieces can be used in a synergistic way or can be used also individually.

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RADIOFREQUENCY: It provides heat to the deeper subcutaneous tissues thanks to the endogenous action of diathermy, without damaging the epidermis. This heat stimulates the production of Collagen, ensuring a progressive and evident push up effect on the skin.

VACUUM ENDOMASSAGE: Performs a deep remodeling action, facilitates the elimination of excess fluid. Specific applicators allow to carry out individually or simultaneously to other energies treatments of detachment, drainage and rhythmic massage.

SWING WAVE: Pressure wave capable of generating a direct mechanical force with the main objective of acting only and exclusively on adipocytes resulting in remodelling of fat deposits and improvement of the appearance of orange peel.

ULTRASOUND 1MHZ: Ultrasounds are waves that propagate at a certain speed based on the density of the tissue.

ULTRASOUND 25 KHZ: Deep cleansing action and delivery of functional cosmeceTics ingredients. The effect of the ultrasound is combined with the vibration exerted by the spatula to perform a mechanical peeling and facial treatments.

MODELING WAVE: The goal of this energy is to reactivate the micro-circulation and promote cell exchanges in areas where adiposity accumulates such as hips, abdomen, knees and thighs.

DERMO INFUSION: Promotes the absorption and enhances the delivery of functional ingredients cosmeceTics selected according to the imperfection to be corrected, allowing him to act directly and immediately.

LLLT LOW LEVEL LASER TREATMENT: The Low Level Laser treatment (930 nm) generates photo-stimulation, capable of improving cellular metabolism, counteracting oxidative stress and inflammation. The skin will appear more toned and compact. Express maximum effectiveness in the treatment adjuvant imperfections of localized fat and cellulite.

NIR NEAR TO INFRARED 855 nm: The NIR (Near to InfraRed) treatment uses infrared capable of reading the chromophore of the water, overheating the tissue, reactivating microcirculation and stimulating the production of new collagen, giving tone and support to the skin.

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Innovative, painless, safe and efficient

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A synergy of different electric waves to act in a targeted way on the most frequent aesthetic blemishes that affect the body. Ideal to awaken the metabolism, improve lymphatic circulation and microcirculation, oxygenate, regenerate and stimulate the production of new Collagen. It redefines the silhouette counteracting the imperfections of fat and cellulite. The skin is more compact and elastic. This laying technology can be defined as an independent operator.

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LLLT LOW LEVEL LASER TREATMENT: The Low Level Laser treatment (930 nm) generates photo-stimulation, capable of improving cellular metabolism, counteracting oxidative stress and inflammation. The skin will appear more toned and compact. Express maximum effectiveness in the treatment adjuvant imperfections of localized fat and cellulite.

ULTRASOUND 1MHZ: Ultrasounds are waves that propagate at a certain speed based on the density of the tissue

TONIC WAVE: Tonic Wave is a technology that stimulates "muscle memory effect" creating filling and firming thanks to the toning of those areas where, for the need of imperfection, reduction is required. Body shapes will be redefined and changes in terms of silhouette remodeling will be noticed immediately.

MODELING WAVE: The goal of this energy is to reactivate the micro-circulation and promote cell exchanges in areas where adiposity accumulates such as hips, abdomen, knees and thighs.

DERMO INFUSION: Promotes the absorption and enhances the delivery of functional ingredients cosmeceutics selected according to the imperfection to be corrected, allowing him to act directly and immediately.

BIPOLAR RF FACE: To regenerate, tone and elasticize the skin of the face, there is also the handpiece of Bipolar Radiofrequency that can be performed simultaneously with the body treatment. A great advantage that translates into a saving of time and greater customer satisfaction, also optimizing the work for the operator.

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Targeted synergies and multiple possibilities

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The pressomassage contributes to the restoration of venous-lymphatic functionality, promoting peripheral circulation. It performs a draining activity, reducing swelling and muscle tension. It has an important detoxifying, revitalizing effect, improves tissue oxygenation, prevents the formation of cellulite blemishes and generates a sense of well-being and lightness. This laying technology can be defined as an independent operator.

BE-PRES | BE-LYMPHOPRES.PRO | BE-LYMPHOPRES.EVO
BE-PRES | BE-LYMPHOPRES.PRO | BE-LYMPHOPRES.EVO

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PULSED MASSAGE: The pulsed inflating action uses a computerized system able to make a pulsed and continuous pressure in every sector. The pulsed and intermittent pressure creates increased stress, improving the effects of the treatment of Presso- body massage. The stimulating action of the subcutaneous tissue improves the contractivity and the muscle metabolism by acting on the reflex route on the arterial circulation and activating a natural drainage mechanism.

SILENT TECHNOLOGY: The technology is equipped with a new generation engine that makes the treatment quieter and the work is efficient over time with a duration of 5,000 hours equal to 10,000 treatments.

EASY LOCK CONNECTION: One of the most common problems in the connection of cables for the at-massage is their insertion not perfect. In this case, the technology could not work, causing inconvenience to customers already ready for treatment. Beautech, which has always listened to the opinions and opinions of Beauticians, has created the Easy Lock connection, a simple but cutting-edge system, which groups all the cables in a single connector with special features, which must be simply connected in the socket on the back of the technology. The tangle of wires will be just a bad memory.

WARM (OPTIONAL): The heat generates pleasant effects and real benefits on the body such as: increased microcirculation resulting in improved interstitial exchanges and increased local metabolism. The increased metabolism has a trophic action, in other words tissue nourishment, but also oxygenation, activity that acts effectively on the imperfections caused by cellulite.

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Innovative technology and specific myotronic massage method developed to obtain concrete results in case of skin and muscle relaxation, loss of tone and elasticity of the face and body. Be-Bodytronic.Pro manages to simultaneously combine Tonic wave muscle stimulation with Radiofrequency stimulation. Thanks to this synergy it is possible to improve the condition of the elastic fibres, oxygenate and tone the muscle mass and reinvigorate microcirculation.

TONIC WAVE | RADIOFREQUENZA | NIR 855 nm | COLLAGENE 630 nm
TONIC WAVE | RADIOFREQUENCY | NIR 855 nm | COLLAGEN 630 nm

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TONIC WAVE: Muscle is a metabolically active tissue, so an increase in muscle tone by electrical stress leads to a parallel increase in basal metabolism. Tonic Wave is a technology that stimulates "muscle memory effect" creating filling and firming thanks to the toning of those areas where, for the need of imperfection, reduction is required. Body shapes will be redefined and changes in terms of silhouette remodeling will be noticed immediately.

RADIOFREQUENCY: It provides heat to the deeper subcutaneous tissues thanks to the endogenous action of diathermy, without damaging the epidermis. This heat stimulates the production of Collagen, ensuring a progressive and evident push up effect on the skin.

NIR NEAR TO INFRARED 855 nm: The NIR (Near to InfraRed) treatment uses infrared capable of reading the chromophore of the water, overheating the tissue, reactivating microcirculation and stimulating the production of new collagen, giving tone and support to the skin.

COLLAGEN 630 nm: Particular emitters at 630 nm are able to stimulate the skin in the production of new Collagen and Elastin.

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Innovative technology for progressive hair removal

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The laser technique is very effective for the treatment of unwanted hair because it is able to slow down its growth until the progressive disappearance. It is a safe, quick and painless technique and can be performed on all parts of face and body. The Beatech laser equipment uses different wavelength, particularly efficient and precise on the hair follicles. The great innovation of this hair removal system is the HD Matrix technology (High Definition Laser) which allows to target the hair bulb in extremely precisely way, due to the fact that the emission of the high definition light beam is much more uniform and does not produce dispersion. The laser wave is absorbed by the melanin, contained into the hair shaft and follicles, it transforms into heat, increases the temperature and triggers a natural physiological process which leads to the progressive disappearance of superfluous hair.

BE-MATRIXHD | BE-MATRIX2HD.PRO | BE-MATRIX3HD.EVO
BE-MATRIXHD | BE-MATRIX2HD.PRO | BE-MATRIX3HD.EVO

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HD MATRIX: The great innovation of this hair removal system is the HD Matrix technology (High Definition Laser) which allows to target the hair bulb in extremely precisely way, due to the fact that the emission of the high definition light beam is much more uniform and does not produce dispersion. The laser wave is absorbed by the melanin, contained into the hair shaft and follicles, it transforms into heat, increases the temperature and triggers a natural physiological process which leads to the progressive disappearance of superfluous hair.

TRIPLE WAVELENGTH: If we consider that, to achieve the optimal result in hair follicle thermolysis, it is necessary to reach a thermal differential between the average body temperature (37°C) and the minimum temperature required to permanently deactivate the cells responsible for hair structure production, and if we consider that for this purpose the traditional 808 nm Diode Laser is already employed with excellent results, it is easy to understand that a simultaneous and balanced synergy between the emission of a 1064 nm Laser beam and bipolar Radiofrequency concentrated on a portion of tissue targeted by the 808 nm Diode Laser beam have a complementary effect in achieving the desired outcome, as they produce a localized increase in temperature, thereby reducing the thermal differential required for the Diode to be effective. This translates into a "thermal advantage" favoring photo-thermolysis, allowing for a more effective modulation of the 808 nm Diode emission, particularly in the treatment of "difficult hairs" such as lighter or thinner ones, making it even more "selective" and thus safer for use on "difficult skin types" like darker phototypes.

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Steam sterilization is the most effective in the world. Safe, quick, simple, environmentally friendly, does not generate high costs or waste. Class B autoclaves are the most advanced steam sterilizers, being certified as medical devices and meeting all sanitary and epidemiological requirements. Class B autoclaves have a vacuum pump that completely removes air from the chamber before starting the sterilization process. This allows steam to penetrate the instruments completely, so it can sterilize all kinds of loads and instruments, even the most complex.

ENBIO BEAUTY EDITION | ENBIO S | ENBIO PRO
ENBIO BEAUTY EDITION | ENBIO S | ENBIO PRO

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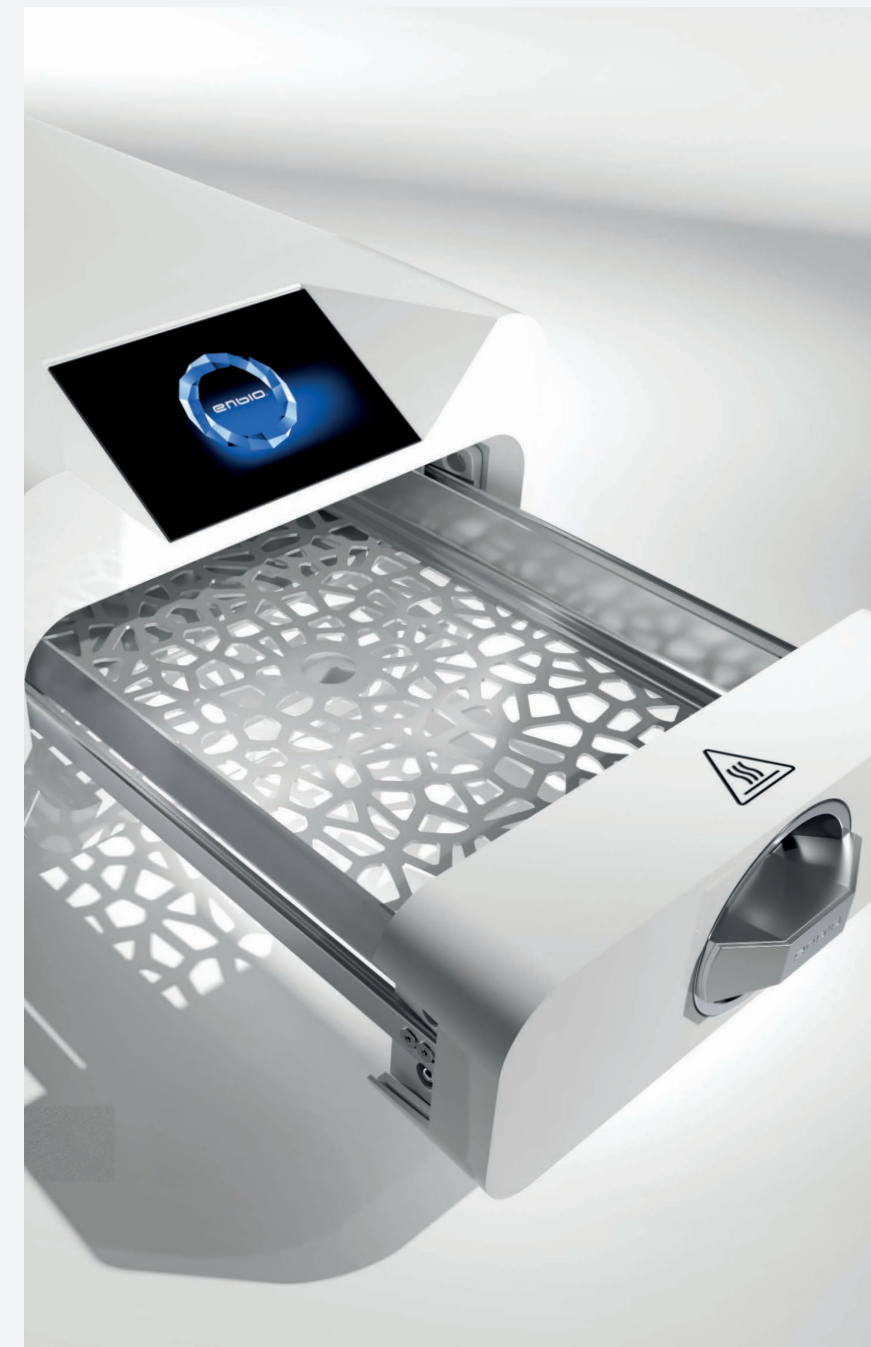
FAST: Enbio is the fastest sterilizer on the market, with a sterilization phase that takes only 4 minutes. Operators can sterilize instruments between one customer and the next or as needed.

SILENT: The noise level generated during the sterilization process is less than a puff, this allows you to work comfortably near the device, sterilizing during the session, without disturbing the tranquility of the environment.

PORTABLE: The device is lightweight and compact, making it easy to move and adapt to any environment. The only thing to consider is to place it not far from an electrical outlet. It is not necessary to connect it to the water supply.

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A health walk in increasing gravity

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Technology to reshape shapes and lose weight in a short time and with little effort. Remodeling of the forms and weight loss in a short time and with little effort. A "health walk" in increasing gravity, which thanks to the combined power of the heat of the infrared rays, delicately increases the temperature of the skin, stimulating the metabolism and distributing the aerobic work on the most difficult areas most difficult to reach with the traditional techniques of weight loss.

Be-Motion.Pro and Be-Motion.Evo are revolutionary device useful for weight loss, increasing metabolism and improving the appearance of the skin not only for experts in physical activity but also for those who do not usually practice it. The presence of a tapis roulant inside the capsule allows to perform a low intensity aerobic physical activity, guaranteeing excellent results with minimum effort.

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VACUUM: A controlled Vacuum effect (dynamic vacuum) is able to simulate a progressive increase of gravity on the body that, applied to the lower limbs, causes an increase in blood flow resulting in improved local circulation.

INFRARED: The combined effect of infrared Fir wave increases tissue metabolism, promotes the opening of the pores and therefore the elimination of toxins, improving the natural outflow of liquids.

OZONE: The emission of ozone ions stimulates the regrowth and revitalization of tissues, reduces the formation of lactic acid, stimulates the metabolism of fat masses, improves circulation and attenuates the "orange peel" appearance.

COLLAGEN: Particular emitters at 620 nm are able to stimulate the skin in the production of new Collagen and Elastin.

AROMA: An automatic dosing station for fresh essential oil fragrances.

MULTIVISION: The Multivision function depicts movement in certain places in the world at walking pace and can be viewed while exercising. You can choose Buenos Aires, Los Angeles, Venice, Costa Rica, Chicago and San Francisco.

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Be-InfraStep.PRO transforms the natural motion of climbing into a dynamic, high-intensity aesthetic treatment. Thanks to a system of continuous steps that simulate stair climbing, the body works against gravity, activating legs, glutes and energy metabolism in a targeted way. Each step stimulates circulation, promotes drainage and increases calorie consumption, helping to redefine the silhouette. The action of movement is enhanced by Infrared heat and Collagen light stimulation, which activate tissues, improve skin oxygenation and support muscle recovery. An advanced workout that combines physical activity, technology and aesthetic wellness, transforming climbing into a complete experience for the body

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Cryosauna is a treatment characterized by exposing the body to a nitrogen cloud for a duration ranging from 1 to 3 minutes. The cryosauna temperature is between -120°C and -170°C . During the treatment, the subject enters Be-Cryo.Pro, the cryochamber, which allows you to leave only the head outside the machine, so that it is not exposed to cold gases. Due to the low temperatures, to which the body is exposed, it is also necessary that hands, feet and private parts are covered with socks, gloves and underwear. At the end of the session, 15 minutes of gentle gymnastics are recommended, which aim to reactivate blood circulation. Instead of gymnastics movement technology such as Be-Motion are also recommended for even more satisfying results.

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EFFECTS ON THE BODY: The basic principle is to lower the body temperature in a very short time. The sudden drop in temperature causes the body extremities to undergo a vasoconstriction, which increases the blood supply to the heart, as well as the relative nutrients and oxygen it contains. On the contrary, the interruption of the exposure to nitrogen, at the end of the treatment, involves a vasodilation, which favors the supply of this concentrated blood of nutrients to the rest of the body. The effects are reflected in a supply of energy nutrients and oxygen to the tissues and in the removal of lactic acid after a workout.

FOR MANY BUT NOT FOR ALL: It is always preferable to undergo treatment after consulting a medical opinion. It will seem impossible to think that the body may be able to withstand a temperature drop below zero and even -170 somebody C. However, the time limitation of the treatment fixed at 3 minutes makes it possible that it does not run into negative consequences. The human epidermis is composed of a very thick layer and, the limited temporal exposure to the source of frost, ensures that the cold can involve only a few millimeters of depth. The skin seems in fact to be able to reach temperatures up to -3 degrees without bearing consequences, while at temperatures below -10 degrees can begin to manifest adverse events such as "cold burns". For this reason, it is essential to adhere to the timing and protocols set by technology and recommended by experienced staff.

NUMEROUS BENEFITS: Cryosauna is known mainly for its numerous applications in sports for muscle and joint recovery; in weight loss for the ability to increase metabolic expenditure. It also has an anti-inflammatory, analgesic and antioxidant action capable of reducing systemic inflammation, pain perception and counteracting the effects of free radicals.

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Be-Circle is the new generation Beautech technology that combines Aesthetics with Fitness for a full body massage and allows specific treatments on the thighs, hips, abdomen, back, calves, stimulating the lymphatic system, counteracting the heaviness of the legs and the imperfections of water retention and cellulite, thanks to the combined action of the Infrared lamp. It is equipped with air ionization, LED touchscreen display and Chromo Wellness. Be-Circle has gained popularity among elite athletes for its ability to accelerate muscle recovery after intensive training, but it is effective and easy to use even for the needs of beginners, giving an unprecedented complete massage experience.

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MASSAGE ROLLERS: The suitably profiled wooden massage rollers allow the full body massage and allow to stimulate the microcirculation, counteracting the imperfections of water retention, cellulite and reshaping the silhouette.

INFRARED: Infrared is the good light of the sun. Its main property consists in heating deep, up to 10-15 millimeters inside the tissues, favoring every metabolism activity of our body. During the movement the Infrared warms the whole body, which draws multiple benefits. The thermal action lasts up to 48 hours after treatment, helping our body to be more receptive and vascularized and, therefore, to eliminate more quickly the toxins accumulated and deposited in the deepest layer of the skin. In addition, the heat makes the massage very pleasant and generates a relaxation of the muscle, ensuring the effectiveness of the treatment.

CHROMO WELLNESS: The treatment of Chromo Wellness uses different colors customizable according to individual preferences, thus achieving greater relaxation and benefit during treatment. Be-Circle coloured lamps can also be operated remotely with a practical remote control.

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Innovative bed for a complete wellness treatment that simulates the absence of gravity thanks to the combined power of three elements: massage, heat and water. The inside located jet nozzles move onto the body with circular and undulating flows of heated water, allow the operator the possibility to select the strength, speed as well as the area of the body where to concentrate the massage according to the individual needs of the customer. The purpose? Cancel the weight. Physical and mental. Floating supported by the water embrace, body and mind will abandon themselves to a pleasant sensation of lightness and contentment.

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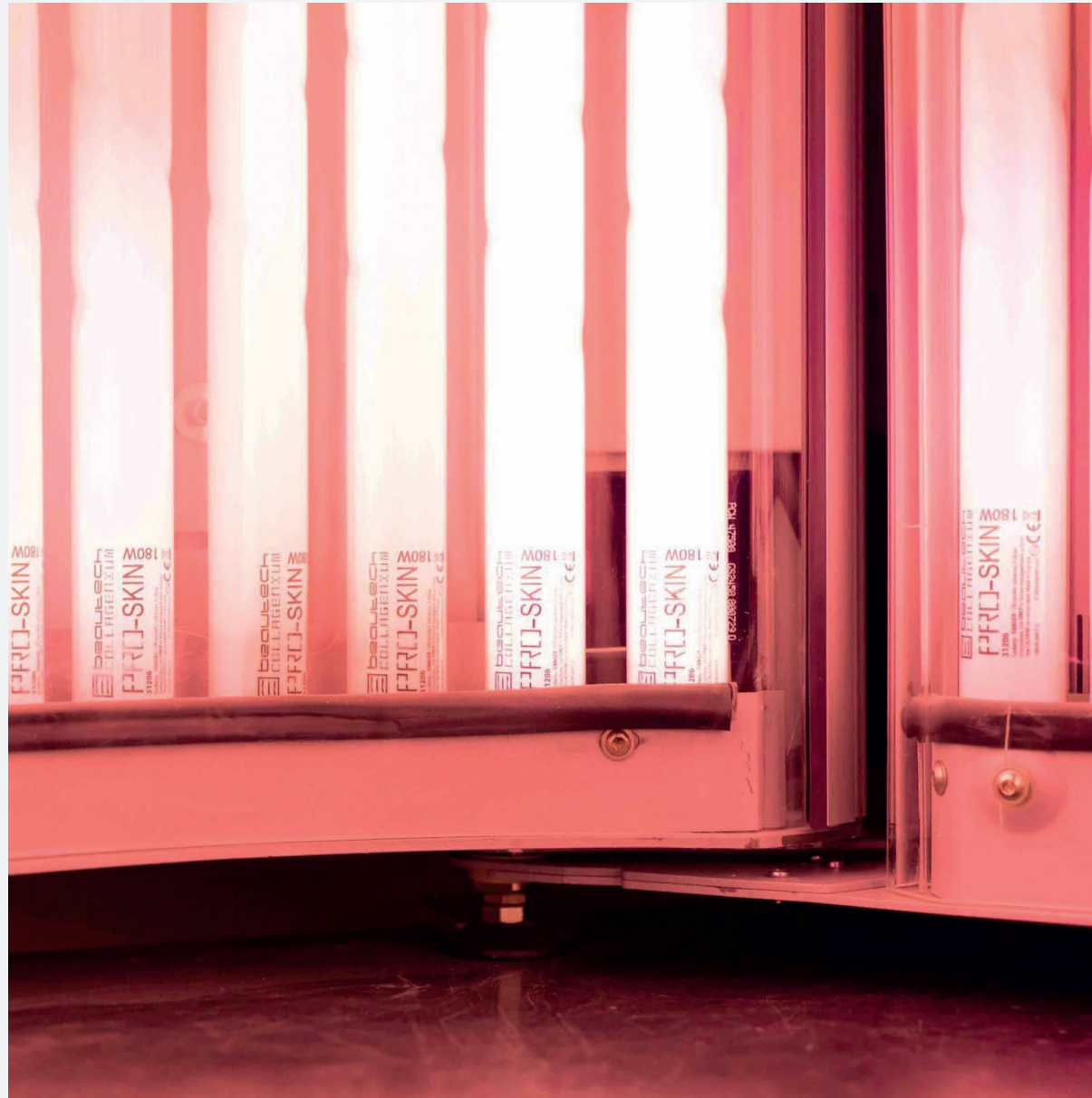
DRY WHIRLPOOL: The effect of the dry whirlpool is the same beneficial effect and feeling of pampering of the real whirlpool but without the need to remove clothing. A specific membrane in fact allows the transfer of the massage without the need for immersion, it will be enough to remove only the footwear.

TOUCH SCREEN DISPLAY: We have equipped Be-Gravity.Pro with a large 10.1 inch touch screen on a special arm that allows you to position the screen towards the operator or the customer. The dedicated software has been designed and created to facilitate its use and to take advantage of all the advantages of the functions that are constantly updated.

WATER TEMPERATURE CONTROL: The software allows you to easily set and manage the water temperature that can be changed during the session. You can also control the power of the jets and change their speed and location.

FACE LIGHT BIO-STIMULATION: The treatment with the chrome mask consists in irradiating, through LEDs, the skin of the face with light of different colors, making the two vibrations (imperfection/ color) interact with each other to obtain an aesthetic advantage and well-being. The chrome mask acts on different skin issues such as impurities, spots or discolorations. It has a significant effect on skin regeneration with signs of aging and age-related collagen loss. It also improves the condition and tension of the skin and soothes any redness.

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Ultra performing solariums both in terms of tanning and anti-aging effects, and at the same time comfortable, fresh, compact and in compliance with European regulations. A complete range of new generation solarium with CollaGenium Inside technology to offer its to customers something truly innovative, differentiating and finally in line with the objectives of aesthetic treatments performed daily in the institutes.

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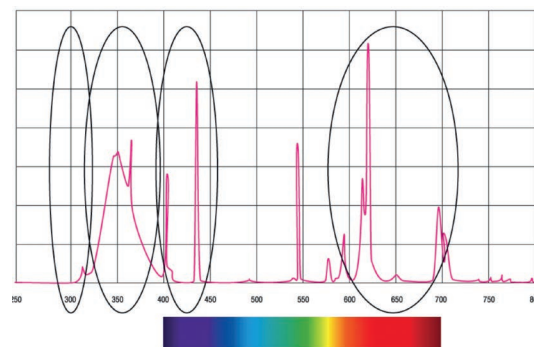
UV-A for an intense, golden and enviable tan: in addition to a visible effect at an aesthetic level, a correct exposure to type UV and A rays stimulates good mood and strengthens the body's immune defenses by increasing the production of endorphins, favors increased concentration and correct metabolism. The Sun is life!

UV-B the stimulates melanin and vitamin D: thanks to this wavelength begins the physiological process of production of new melanin, the natural pigment of the skin, but not only, is the only way that the body can process and assimilate vitamin D, essential for important vital functions such as the fixing of Calcium, a process that allows to strengthen the bones (counteracting the pathologies that weaken them) and produce keratin for hair and nails. The Sun is health!

LIGHT BLUE (430 nm) for flawless skin: this particular frequency of light is already known in the aesthetic and medical field for years for its marked antiseptic function capable of assisting treatments aimed at imperfections with the presence of bacterial forms such as acne, also has the ability to reduce the occurrence of stains often present on the skin. The Sun is perfection!

RED LIGHT (633 nM) collagen for always young and fresh skin: the photobiostimulation generated by the frequency of the red color of these tubes effectively stimulates the fibroblasts present in the dermis, the organs predisposed to the production of new collagen. In this way the skin is, already after a few sessions, more toned and vital, firmed and luminous, thus ensuring a real anti-age function. The Sun is youth!

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CollaGenium technology is compatible with most solarium models in use and complies with the standards of the 0.3 W/M² legislation in force.

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Funzioni	Aesthetic Pricking®
	Nano Advanced
	Micro Pigmentation
	Trico Pigmentation
	Areola Pigmentation

- | | |
|-------------------------|-------------------------------|
| ○ 1PM 0.18 - NANO LINER | ● 5 MM 0.25 - MAGNUM |
| ● 3RS 0.25 - SHADER | ● 130AN - NANO PEEL |
| ● 5QM 0.25 - LINER | ● 9MP 0.18 AESTHETIC PRICKING |

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Funzioni	RF Need System
	Aesthetic Pricking®
	Nano Advanced
	Micro Pigmentation
	Trico Pigmentation
	Areola Pigmentation

- | | |
|-------------------------|-------------------------------|
| ○ 1PM 0.18 - NANO LINER | ● 130AN - NANO PEEL |
| ● 3RS 0.25 - SHADER | ● 9MP 0.18 AESTHETIC PRICKING |
| ● 5QM 0.25 - LINER | ● RF9MP 0.18 - RF NEED SYSTEM |
| ● 5 MM 0.25 - MAGNUM | |

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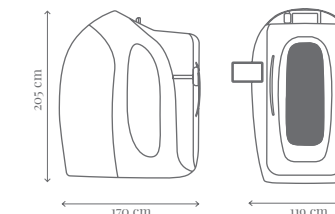
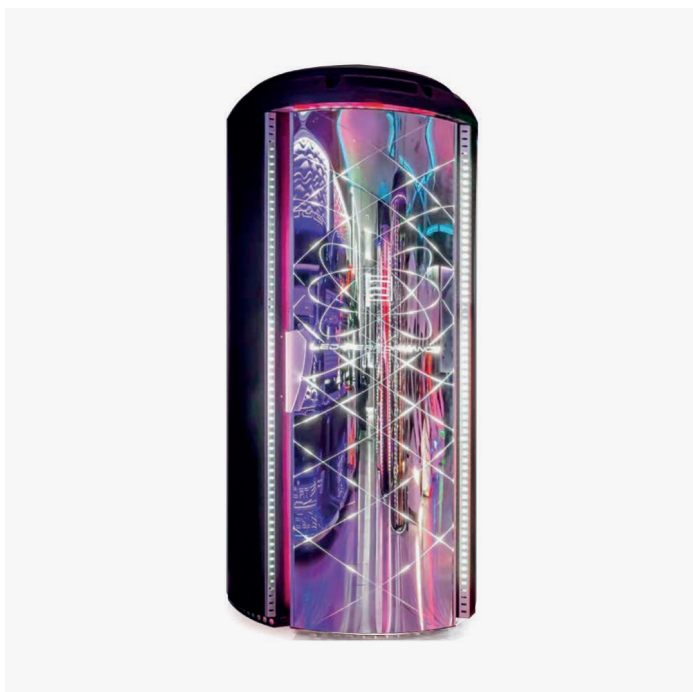
COLLAGENIUM HYBRID TOWER / COLLAGENIUM INTENSIVE TOWER

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COLLAGENIUM INTENSIVE BED / COLLAGENIUM HYBRID BED

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Dotazioni	Acqua Breeze
	Sound System
	Digital Display
	Antipanic
	Ambient Led Lights
Optional	Open/close electronic system
	Bluetooth
	Conditioned air
	Timer preaccensione
	Wireless smartphone charger
	Chip card system

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The solution for the digital beautician

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Complete management software that allows you to organize every aspect of the Beauty Center in order to optimize time and increase sales. Digital planning replaces the traditional paper diary, allowing appointments to be recorded in real time. By clicking on the appointment you will see everything relating to the treatment in detail: customer details, operator, date and time, financial situation in detail and much more. Be-Solution allows you to keep the economic situation under control by analyzing the day's reports thanks to the possibility of easily viewing the proceeds divided by treatments, products, subscriptions and operators. The management system is designed to make reservations and sales online.

**SOFTWARE GESTIONALE IN CLOUD
MANAGEMENT SOFTWARE IN CLOUD BE-SOLUTION**

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DIGITAL PLANNING: With a simple tablet, every beautician can have their appointments under control, fill out the medical records and the customer registry.

PERSONAL DETAILS: An advanced address book with all customer data: name, surname, phone, date of birth, address, e-mail, etc...

STATISTICS: Check employee productivity, how much and what each customer spends, view daily reports and more.

DATA SHEET: for a total history of each physical and aesthetic characteristics of the customer: imperfections, measurements and photos before/after.

CARD MANAGEMENT: With a simple barcode reader you can manage fidelity cards, gift cards and credit cards.

WAREHOUSE: Records sales to keep under control the internal consumption of products directly from the management.

ONLINE BOOKING: Sell all treatments 24 hours a day, 7 days a week, eliminating the loss of last minute cancellations.

DIGITAL RECEIPT: Be-Solution is compatible with all telematic checkouts and makes daily use much easier.

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A personalized application from the beauty center to stay in constant contact with their clients, who will be able to schedule their appointments 24 hours a day, view their appointment calendar in real time, and check their subscriptions. Additionally, it's possible to send notifications directly to the phones of all clients to communicate various initiatives.

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Looking to the future. A modern and functional website with proper visibility on the web, an updated communication and modeled according to the latest indexing standards, dynamic thanks to social interaction and your virtual shop, open 24 hours a day and 7 days a week, can provide the traditional business with additional profit as well as new customers.

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A team of experts helps and guides in managing communication through the most effective promotional channel in the world, creating commercial campaigns on social channels aimed at developing contacts and appointments with new customers during promotional event days.

SOCIAL MEDIA MARKETING | LEAD GENERATION
SOCIAL MEDIA MARKETING | LEAD GENERATION

BEAUTECH

BEAUTECH S.R.L.
Via Omero, 51
41123 Modena (MO), Italia
+39 059 468349
direct@beautech.it
www.beautech.it

BEUTECH